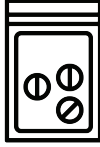



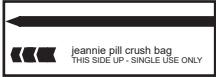





# Operating Instructions

**1.**  Always use a NEW POUCH for each use.

**2.**  Insert pills into bag  
(RECOMMEND MAX 6 PER BAG)

$\text{O} \times 6$  Pills must be flat  
Arrange side by side 


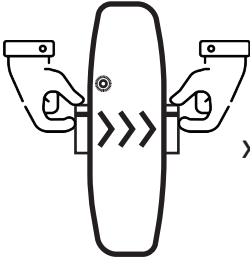
**3.**   Make sure pouch has writing side facing up and arrows pointing towards machine in the same direction as the arrows on the machine.

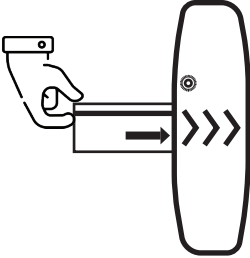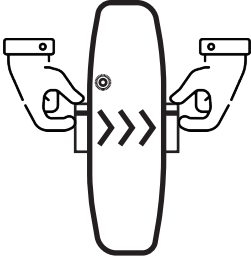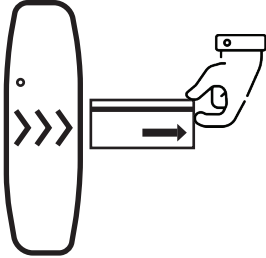**4.**   


**INSERT POUCH INTO MACHINE.**  
(Crusher starts automatically)  
(Indicator light comes on)

**TAKE OTHER SIDE WITH OPPOSITE HAND.**  
(Pull pouch through SLOWLY to avoid machine jamming or damaging pill pouch)

**PULL THROUGH GENTLY UNTIL CLEARS MACHINE.**  
(Usually takes 3 to 5 seconds)  
(Indicator light will turn off  $\pm 10$  seconds after crush completed)

**5.**   x2 | x3

If a finer powder consistency is required, the pill pouch can be passed through the machine again.

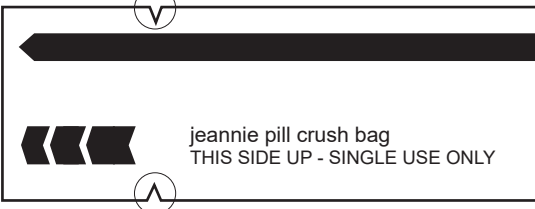**6.** 

The pill pouches have pre-cut notches to allow for easy opening and dispensing of product.




## How to Charge

**1.**




Plug Jeannie charger into wall socket and make sure wall plug is on.




Indicator light on charger will show GREEN to confirm charger is receiving power. (When not connected to Jeannie pill crusher unit.)

**2.**




Ensure switch on underside of Jeannie pill crusher is switched to "ON".

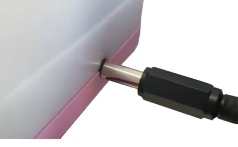

Plug charger into Jeannie pill crusher unit.

**3.**




Ensure charger plug is securely pushed all the way into the socket on the unit.




Indicator light on charger will show RED to confirm charger is busy charging.

**4.**

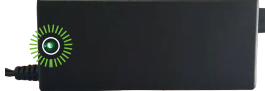


When unit is fully charged, the indicator light on the charger will show GREEN.

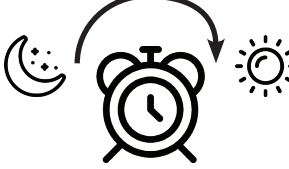

It is recommended that you charge the Jeannie overnight ready for the next day's use.


## Indicator Light Explanation



Indicator light on unit glowing constantly

Indicates Jeannie is IN OPERATION


Your Jeannie is functioning correctly. The light will turn off after a few seconds if the machine remains idle. (Keep the entrance slot free from dust and liquids. Wipe exterior with clean damp cloth. Compressed air can be used to blow out dust.)



Indicator light on unit repeatedly shows a single flash

Indicates Jeannie has LOW OR FLAT BATTERY

Put Jeannie Pill Crusher on charge. (Unit Indicator light will turn off when charging) Jeannie can be used while charging. (NOTE: Jeannie may still continue to function when you see a constant single flash as the battery is low, but not yet completely flat.)

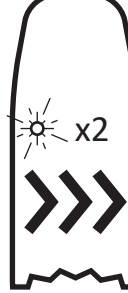

Indicator light on unit repeatedly shows a double flash


Indicates Jeannie is JAMMED

Crusher motor has stopped during operation due to being jammed.


**Reset the machine:**

- By placing a new bag in the entrance slot OR
- Turning the machine OFF for 5 seconds then turning ON again.

## Removing a Jam

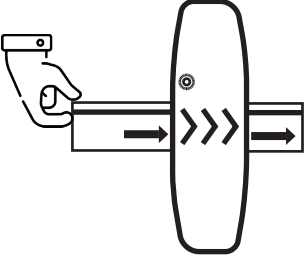

If a pouch gets stuck in the unit, you can remove it by: restarting the crusher by placing a new bag in the entrance slot to activate the motor, which may allow you to pull the jammed pouch out. Once the jammed pouch is removed, reset the machine.

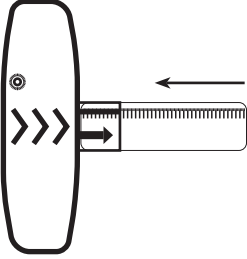

If reactivating the motor is still not effective, a ruler (or similar) can be used to push the pouch out in the direction opposite to normal operation (i.e. towards the start) and removed from the entrance slot. Once the jammed pouch is removed, reset the machine.