

# INVACARE® MATRX® CLINICAL SEATING & POSITIONING GUIDE



Clear and comprehensive clinical tool that highlights the most commonly seen postural asymmetries with potential solutions designed for function and comfort.

Retired Marine Corps Staff Sgt. Ronnie Jeffrey Jimenez is a medal-winning hand-cyclist and swimmer. He encourages fellow disabled veterans to give adaptive sports a try. "It's great to be able to compete again and have fun," Staff Sgt. Jimenez said. "You can do anything your heart desires despite your injury, disability or sickness." When he isn't competing, Staff Sgt. Jimenez stays active using his küschall<sup>®</sup> K-Series *attract*<sup>™</sup> Wheelchair with Invacare<sup>®</sup> Matrx<sup>®</sup> Libra<sup>™</sup> cushion and MX2 Back.



# Quick Reference Overview

# TABLE OF CONTENTS

Quick Reference - Overview	PAGE 5
External Forces - Pressure, Shear, Compression, Friction, Temp & Moisture	PAGE 6-7
Cushion Design Features - Immersion, Off-loading, Envelopment	PAGE 8-9
Cushion Design Features - Shear Management, Pelvic Stability	PAGE 10-11
Posterior Pelvic Tilt with Kyphosis and Clinical Presentation	PAGE 14-15
Adjustability - Wedges, Abductor, Supplemental Fluid Sac	PAGE 12-13
Potential Solutions (Flexible and Fixed)	PAGE 16-17
Anterior Pelvic Tilt with Hyperlordosis and Clinical Presentation	PAGE 18-19
Potential Solutions (Flexible and Fixed)	PAGE 20-21
Pelvic Obliquity and Scoliosis and Clinical Presentation	PAGE 22-23
Potential Solutions (Flexible and Fixed)	PAGE 24-25
Pelvic Rotation and Clinical Presentation	PAGE 26-27
Potential Solutions (Flexible and Fixed)	PAGE 28-29
Hip Abduction	PAGE 30
Potential Solutions (Flexible and Fixed)	PAGE 31



She is currently working toward her Bachelor's in Psychology and enjoys modeling, skiing, kayaking, and painting in her free time. Stephanie is very active in her community with her fundraising efforts. "My community has done so much for me and I strive to give back!" she said.

#### QUICK REFERENCE - OVERVIEW

#### INVACARE MATRX SEATING OBJECTIVES

- Facilitate postural stability while allowing purposeful movement to promote effective function and support healthy resting postures.
- Respect 3 dimensional anatomical shapes, working to match contours for optimal support and pressure redistribution.
- Wherever possible, support postures from within the contours of the seating system in order to maintain skin integrity and to promote stability, balance and function, complemented with additional external components as needed.

#### INVACARE MATRX GUIDING PRINCIPLES

- The effect that seated posture has on breathing and swallowing should be a primary concern.
- Long term sitting can cause secondary complications such as tissue trauma, back and neck pain, postural deformities and joint contractures.
- A comprehensive evaluation, including a physical assessment in both supine and sitting, is the foundation of all effective seating solutions.
- The position of the pelvis directly impacts the spine, which in turn influences the position of the head and extremities.
- The pelvis is the foundation for seated function and the PSIS (posterior superior iliac spine) must be supported in order to achieve postural control.
- Determining if a posture is fixed or flexible is vital for selecting appropriate seating solutions.
- The opportunity to trial seating solutions in static and dynamic situations is important for identifying the most effective overall seating solution.

The information provided in this clinical seating and positioning guide is the opinions of clinicial staff at Invacare Corporation and proper assessments should be made at the individual patient level. This information is not intended to be, nor should it be considered, medical, billing or legal advice. The physician and other medical care providers are responsible for determining proper product selection and the appropriate billing codes when submitting claims to the Medicare program, and should consult an attorney or other advisor to discuss specific situations in further detail.

# EXTERNAL FORCES



PRESSURE A vertical force that occurs at a 90° angle.



#### SHEAR

Parallel forces, sometimes described as stretching forces occurring within the tissues near bony prominences.



#### COMPRESSION

Postural collapse that can impede the function of the internal organs.

### EXTERNAL FORCES



#### FRICTION

The resistance that arises when one surface rubs against another.

#### TEMPERATURE/MOISTURE

Raised body temperature and higher humidity that can contribute to tissue damage.

# CUSHION DESIGN FEATURES





#### **IMMERSION**

Maximizes surface contact area in order to reduce peak pressures. Pressure = Force/Area.

#### OFF-LOADING

Transfers forces away from the ischial tuberosities with greater weight bearing on the trochanters, hips and thighs.

# MATRX

#### ENVELOPMENT

Fluid sac completely surrounds the bony prominences, even during position changes.

#### MATRX



#### SHEAR MANAGEMENT

Static: Support surface contour that prevents sliding and downward migration of the pelvis. Dynamic: Flow of materials surrounding the bony prominences during functional activities, wheelchair propulsion, accommodating movement within a specific range.

#### MATRX - UNIQUE DESIGN



#### PELVIC STABILITY

Contouring of the pelvic well provides lateral trochanteric support and anterior/ posterior pelvic stability, promoting spinal extension for improved upper extremity function, head control and physiological function. Adjustable rigid back support with contact at PSIS to maintain optimal pelvic position.



#### MATRX



#### ANTERIOR (FRONT) WEDGE(S)

- Use on both sides to help prevent sliding
- Use on one side only to facilitate foot contact on the low side and foot support on the high side eg. Hemiplegic client
- Can be used to accommodate hip flexion limitation

#### TAPERED WEDGE(S)

- Use under front corner to encourage adduction
- Use under rear corner to address pelvic obliquity

#### MATRX



# LATERAL (SIDE) WEDGE(S)

- Can be used to create deeper contour





ABDUCTOR - Promotes abduction of thighs

#### SUPPLEMENTAL FLUID SAC

- Secure by threading on to lateral strap of fluid overlay and positioning on trachanteric shelf (not in well)
- Can be used to address pelvic obliquity, alone or in combination with tapered wedge, or bilaterally to create deeper contour

# POSTERIOR PELVIC TILT WITH KYPHOSIS

#### POTENTIAL CAUSES

Wheelchair Fit Seat depth too long/short Foot support position No support for PSIS In manual wheelchair, location of rear wheel not optimal for reach Seat-to-floor height too high/low for foot propulsion

#### Clinical

High or low tone in trunk Lacks true 90 degrees of hip flexion Weak abdominals/back extensors Shortened/tight hamstrings Assumes position for increased postural stability

#### CLINICAL PRESENTATION



Posterior Pelvic Tilt

Cervical spine may hyper-extend to maintain a functional visual field

Scapula may protrude posteriorly

Flattened Lumbar Spine and increased Thoracic Kyphosis ("C" shape Thoracic curve)

Tendency for pelvis to slide forward

Pelvis tilted posteriorly with the Anterior Superior Iliac Spine (ASIS) higher than the Posterior Superior Iliac Spine (PSIS)

# POSTERIOR PELVIC TILT WITH KYPHOSIS

#### **POTENTIAL SOLUTION -** FLEXIBLE POSTURE

Consider anterior chest support

Consider postural support tray

Trial contoured cushion to help stabilize the pelvis in a neutral position

Firm back support that stabilizes from the PSIS up to slightly above the apex of the kyphotic curve

Place pelvic positioning belt beneath the ASIS with line of attachment below the pelvis 45-60 degrees

#### **POTENTIAL SOLUTION - FIXED POSTURE**

Use tilt to achieve a more upright position and improve functional visual field/head position

> Contoured cushion to support the pelvis

Provide a contoured back support to match shape of spine

If spinal process or ribs are prominent, increase backrest depth for greater immersion

For a unilateral limitation in hip flexion, modify the cushion to split the sagittal seat angle to accommodate and try to achieve a level pelvis

# ANTERIOR PELVIC TILT WITH HYPERLORDOSIS

#### POTENTIAL CAUSES

#### Wheelchair Fit

Back support too upright Excessive lumbar contouring

#### Clinical

Tight quadriceps/hip flexors/paraspinal muscles Weak abdominal musculature Obesity

#### **CLINICAL** PRESENTATION

Trunk often extended to compensate for instability from anterior pelvic tilt

Pelvic to thigh angle less than 90 degrees

May present with shoulder retraction

May present as exaggerated lumbar lordosis and result in decreased contact with the back support surface

Pelvis tilted anteriorly (forward) with ASIS lower than the PSIS

# ANTERIOR PELVIC TILT WITH HYPERLORDOSIS

#### **POTENTIAL SOLUTION -** FLEXIBLE POSTURE

Possible anterior trunk support

Early use of powered seating to allow independent adjustment for comfort and/or function balance

Each client will differ in preference of sagittal seat and back support angles, especially those with Muscular Dystrophy and Spina Bifida

Assess small incremental changes to seat slope, angle adjustment of wheelchair back frame and/or the angle orientation of the back support to move the pelvis and spine into a neutral orientation

Provide back support at level of PSIS to reduce lordosis

#### **POTENTIAL SOLUTION - FIXED POSTURE**





Lumbo-sacral insert can be added to increase contact with the spine

Angle adjustable back support

Angle rear of the seat lower than the front to balance trunk over the pelvis

# PELVIC OBLIQUITY AND SCOLIOSIS

#### POTENTIAL CAUSES

#### Wheelchair Fit

Sling or stretched seat upholstery

Seat width too wide and/or arm supports too low to support upper extremities

Cushion does not provide effective support for greater trochanters

Wheelchair seating angles and/or foot support position does not accommodate hip range limitations Power wheelchair joystick or manual wheelchair rear wheel location not optimal for reach Back support too wide

#### Clinical

Asymmetrical pain or discomfort Asymmetrical Tonic Neck Reflex (ATNR) Asymmetrical muscle tone/weakness in trunk and/or lower extremities Limitations of hip flexion, abduction, adduction, internal or external rotation Structural bony deformity in spine or surgery Asymmetrical upper extremity strength with manual propulsion

#### **CLINICAL** PRESENTATION

The obliquity is referred to by the lower side of the pelvis. The spine is influenced by the oblique pelvis, resulting in a scoliosis. The spinal curve will be convex on the oblique (lower) side of the pelvis  One shoulder often elevated on the oblique side lower/weight bearing side

> Increased risk of pressure ulcer on the oblique (lower) side

 One side of the pelvis is lower then the other notably at the ASIS or PSIS

# PELVIC OBLIQUITY AND SCOLIOSIS

#### POTENTIAL SOLUTION - FLEXIBLE POSTURE



#### **POTENTIAL SOLUTION -** FIXED POSTURE

Ensure lateral depth is deep enough to support the lateral trunk

If fixed, build up support under the highest ischial tuberosity to increase weight bearing on high side



Foam in Place Kit may be used for additional support for mild to moderate postural asymmetries

A contoured back with integral lateral support

# PELVIC ROTATION

#### POTENTIAL CAUSES

#### Wheelchair Fit

Poor wheel placement on manual chair Seat-to-floor height too high for foot propulsion

#### Clinical

Limited hip flexion, abduction, adduction Leg length discrepancy may be caused by dislocated or subluxed hip Unequal buttock/thigh depth, leg length discrepancy

#### **CLINICAL** PRESENTATION

One ASIS and therefore hip is further forward in the seat than the other



One hip is abducted and the

 One knee may appear further forward in the presence of a leg length discrepancy or dislocated

# PELVIC ROTATION

#### **POTENTIAL SOLUTION -** FLEXIBLE POSTURE

Align pelvis in neutral and accommodate asymmetrical lower extremity posture

Use a pelvic positioning belt to bring hips back in alignment

In order to maintain head and shoulder in a neutral position for function, you may need to maintain some asymmetry in the pelvis. In this case, use an anterior trunk support on the forward side

#### **POTENTIAL SOLUTION -** FIXED POSTURE

Contoured back support rotated to accommodate any mild to moderate trunk rotation to support the spine and distribute pressure



Accommodate limited hip flexion by opening seat to back angle

 If present, measure the leg length difference.
 Order cushion for longer leg length and specify amount to cut-back on shorter side

# HIP ABDUCTION

#### POTENTIAL CAUSES

Low or high tone Surgeries, LE Abduction due to excessive abdominal tissue Inadequate seat depth



Movement of the femur away from midline Can be unilateral or bilateral

Lower extremities are separated further apart from neutral

# HIP ADDUCTION

#### POTENTIAL CAUSES

Sling upholstery without solid seat insert Low or high tone, decreased range of motion and/or strength of hip abductors



Movement of the femur toward the midline Can be unilateral or bilateral

#### **HIP ABDUCTION**

Flexible: Try to align femurs in neutral using contoured cushion

Try distal lateral thigh supports

Fixed: Accomodate with custom contoured seating

#### **HIP ADDUCTION**

Flexible: Use seat ridgidizer or solid seat pan Try distal medial thigh support or contoured seating

Fixed: Accommodate with custom contoured seating





Contoured Cushion to align lower extremities



Cushion ridgidizer contoured to eliminate "hammock effect" of sling upholstery







 WA
 (08) 9456 1661
 VIC
 (03) 9580 1055
 SA
 (08) 7225 8720
 QLD
 (07) 3448 9243
 NSW 1300 467 737

 www.hospequip.com.au
 www.hospequip.com.au
 www.hospequip.com.au
 www.hospequip.com.au
 NSW 1300 467 737